

5 Steps to Success:

1



Wash your hands.

2



Scrub 30 seconds.
Dry 2 minutes.
Do not palpate again.

3



Clean bottle tops with
alcohol. Let dry.

4



Draw 20 mL:
10 mL in green,
remainder in purple.

5



Label bottles with:
collection site, time,
date, initials.

5 Steps to Success:

1



Wash your hands.

2



Scrub 30 seconds.
Dry 2 minutes.
Do not palpate again.

3



Clean bottle tops with
alcohol. Let dry.

4



Draw 20 mL:
10 mL in green,
remainder in purple.

5



Label bottles with:
collection site, time,
date, initials.

5 Steps to Success:

1



Wash your hands.

2



Scrub 30 seconds.
Dry 2 minutes.
Do not palpate again.

3



Clean bottle tops with
alcohol. Let dry.

4



Draw 20 mL:
10 mL in green,
remainder in purple.

5



Label bottles with:
collection site, time,
date, initials.

5 Steps to Success:

1



Wash your hands.

2



Scrub 30 seconds.
Dry 2 minutes.
Do not palpate again.

3



Clean bottle tops with
alcohol. Let dry.

4



Draw 20 mL:
10 mL in green,
remainder in purple.

5



Label bottles with:
collection site, time,
date, initials.