



RO3 REVERSE OSMOSIS WITH DEIONIZER 185 GAL STORAGE TANK AND HIGH FLOW PUMP MODEL 00HCS-2055

Healthcare Water Treatment Systems Technical Data Sheet

DESCRIPTION:

Provides low conductivity water for washers, disinfectors, glassware washers, steam boilers and final rinses. Reverse osmosis system, a deionizer polisher, and a storage tank with demand pump package that provides up to 4,200 GPD.

FEATURES:

- Pre-carbon block filter
- Anti-scalant feed to prevent hard water scale on reverse osmosis
- Controller with conductivity of feed and product water
- Deionizer polisher includes a 200,000 ohm-cm monitor with alarm
- Simple disinfection cycle
- Storage tank with sealed lid and pump package
- Low feed pressure system shutdown / alarm
- High conductivity alarm
- Storage tank empty pump shutdown


STANDARDS:

- ETL listed, conforms to UL Std 61010-1
- Certified CAN/CSA C22.2 No. 61010-1
- ISO134585 registered device
- Seismic OPA-2202-07



PRINCIPLE OF OPERATION for COMPLETE WATER TREATMENT SYSTEM:

Tap water enters the system through the filters, after which the anti-scalant chemical is pumped into the flow to the reverse osmosis pump and membranes. DI product water flows to the 185 gallon storage tank, reject concentrated waste water flows to the drain (product and reject are 50% each). The demand pump runs when flow is needed and pumps the low conductivity water to the use.

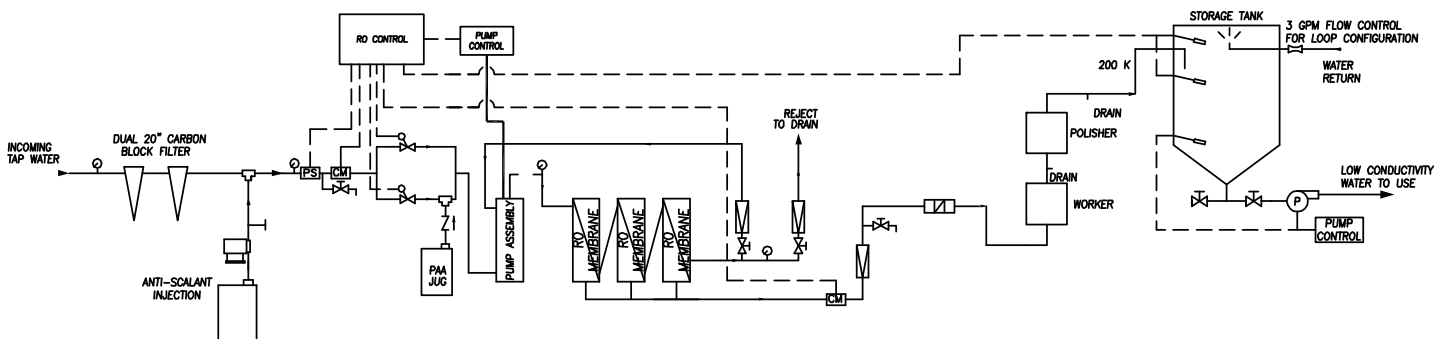


STERIS Corporation
5960 Heisley Road
Mentor, OH 44060-1834 • USA
440-354-2600 • 800-548-4873
www.steris.com

**RO3 REVERSE OSMOSIS WITH DEIONIZER
185 GAL STORAGE TANK AND HIGH FLOW PUMP
MODEL 00HCS-2055**

**Healthcare Water Treatment Systems
Technical Data Sheet**

SPECIFICATIONS	
Drain	One 1-1/2" floor drain or sink
Feed Water Quality	Potable drinking water standards
Feed Water Temperature	Between 41°F - 90°F (Ideal 77°F)
Feed Water Flow Minimum	5 GPM
Feed Water Pressure Min. / Max.	20 PSI / 50 PSI
Projected Rejection	> 96%
Production Rated Capacity*	4,200 GPD / 15,897 LPD
Storage Tank (Gallons)	185
Distribution Pump Output	30 GPM @ 30 PSI
Electrical	One 115V, 15 amps dedicated circuit Two 220V, 15 amps dedicated circuit
Recommended Space In. (W x D)	129 X 72
System Dimensions In. (W x D x H)	105 X 36 X 94
Weight – Shipping / Operating Lbs.	893 / 2,438

* Product flow rate varies with temperature and inlet pressure. All models are rated at 77°F with feed water of 1500 mg/l NaCl @ 55 PSI feed pressure and pH of 7.5. As a safety factor use 60% of the rated capacity when sizing or add blend valve to temper the feed water.



INSTALLATION:

The system can be installed with up to 50' of pipe from the use.

ACCESSORIES	
00850300	Remote alarm package
75679117	1 megohm monitor with alarm for deionizer
00HC-5001	Blend valve assembly

WARRANTY:

One year on parts.

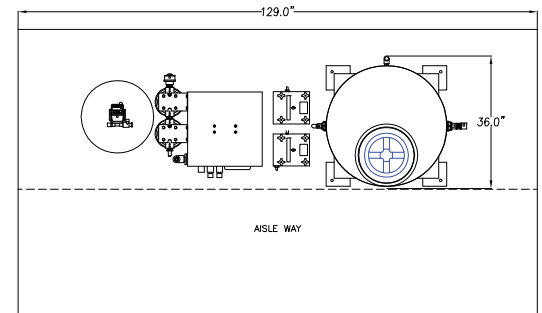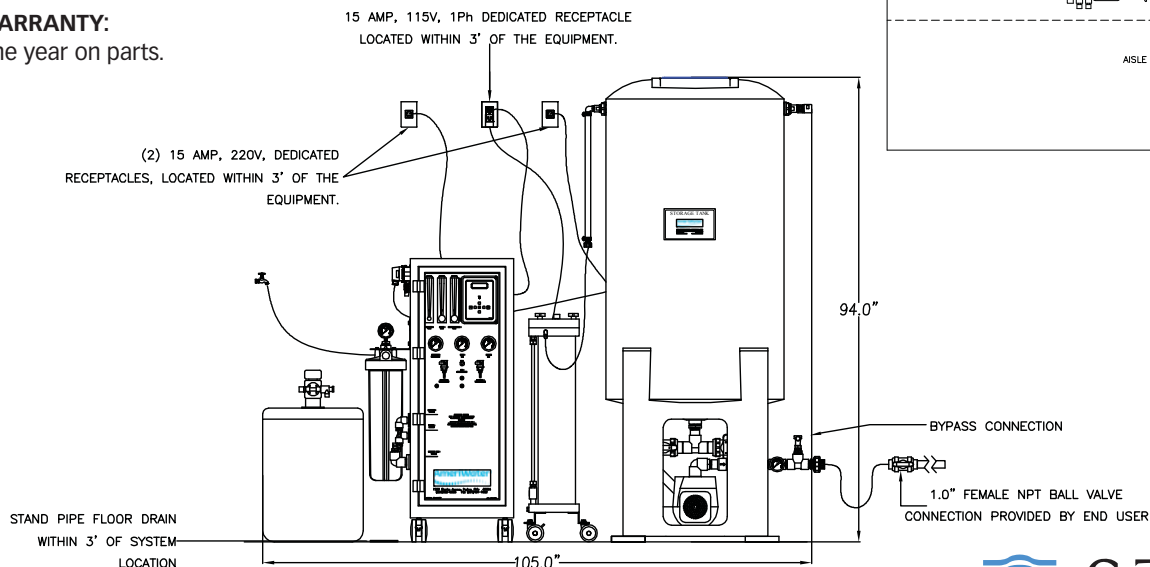

STERIS Corporation
5960 Heisley Road
Mentor, OH 44060-1834 • USA
440-354-2600 • 800-548-4873
www.steris.com