


TYPICAL WALL SUPPORT ELEVATIONS


S71

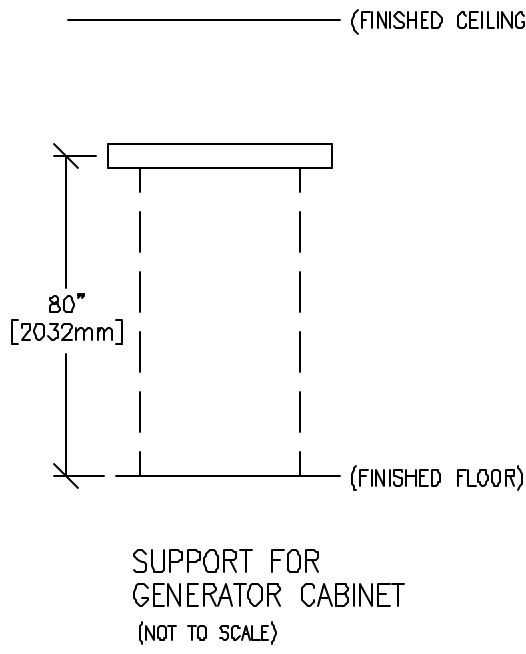

SCALE: 1/4" = 1'-0"

STRUCTURAL LAYOUT

EXISTING CEILING HEIGHT = 9'-2"

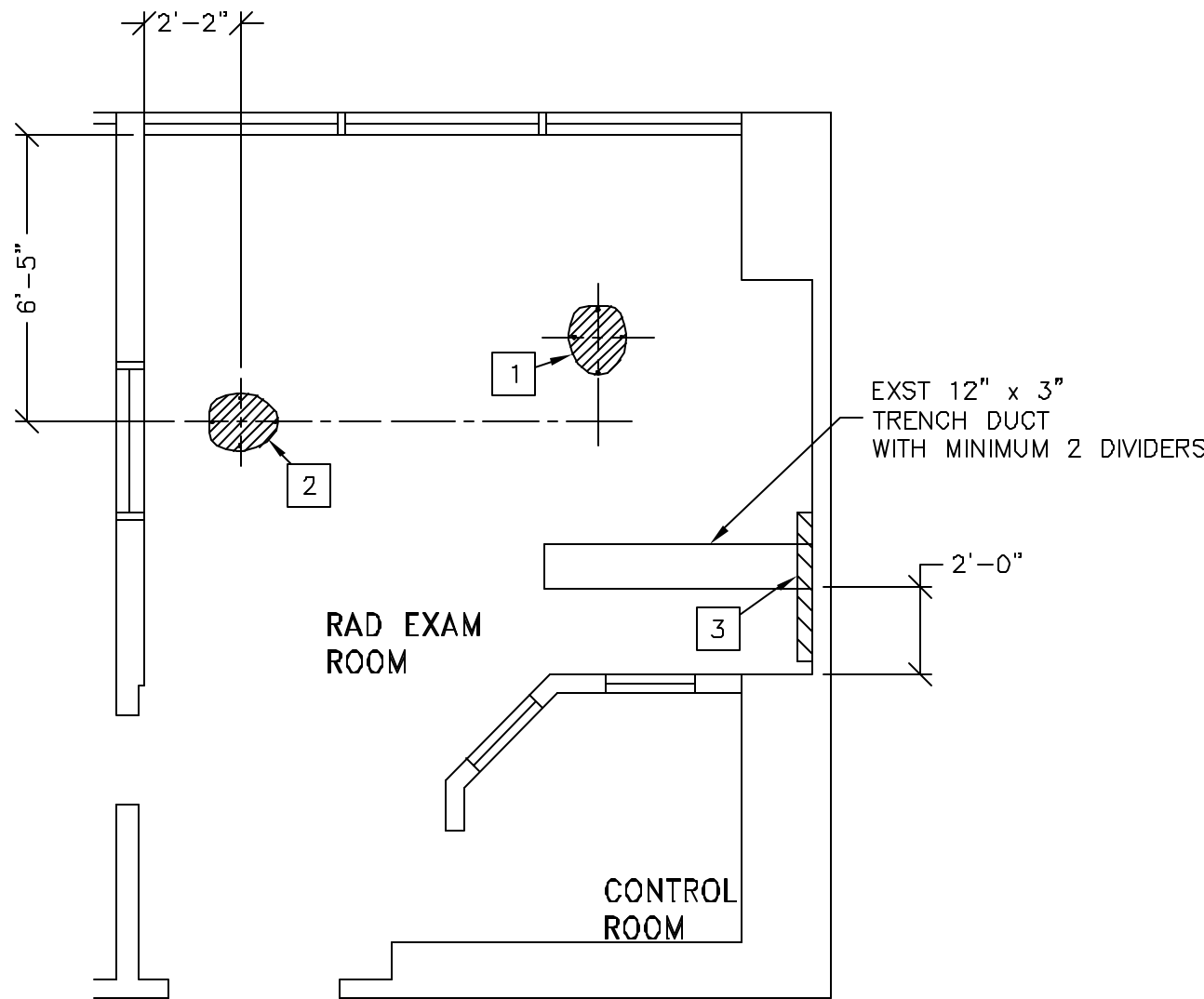

Drawn by: CHRIS CHRISTOPHERSON Octel no.: 5603729
GE Installation
Project Manager: RANDY HAY
Telephone no.: (516) 987-7302

STRUCTURAL SUPPORT METHODS

CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS

ITEM NO.	ITEM DESCRIPTION (* INDICATES EXISTING)
<input type="checkbox"/>	
1	FLOOR CONTACT AREA FOR TUBE STAND
2	FLOOR CONTACT AREA FOR CHEST READER
3	SUPPORT BACKING, LOCATE AS SHOWN, REFER TO ELEVATION DETAIL S71, FOR GENERATOR CABINET

STRUCTURAL NOTES

- ALL STEEL WORK AND PARTS NECESSARY TO SUPPORT CEILING MOUNTED TUBE HANGER OR OTHER EQUIPMENT ARE TO BE SUPPLIED BY THE CUSTOMER OR HIS CONTRACTORS. THE UNISTRUT OR EQUIVALENT STRUCTURE SHOULD RUN CONTINUOUS WITH NO FITTINGS EXTENDING BELOW FACE OF UNISTRUT CHANNEL, RUN WALL TO WALL, BE PARALLEL, SQUARE AND IN THE SAME HORIZONTAL PLANE FLUSH WITH FINISHED CEILING. THE SYSTEM IS TO BE CROSS BRACED VERTICALLY, HORIZONTALLY AND DIAGONALLY TO ALLOW NO MOVEMENT AND A MAXIMUM OF 1,58mm(1/16") DEFLECTION. (10) 12,7mm (1/2") DIA. x 38,1mm (1 1/2") LONG BOLTS WITH UNISTRUT 12,7mm (1/2") NUTS WITH SPRINGS ARE TO BE PROVIDED BY CUSTOMER OR HIS CONTRACTORS FOR EACH STATIONARY AND AUXILIARY SUPPORT RAIL. CLOSURE STRIPS SHALL BE PROVIDED FOR AREAS OF UNISTRUT EXPOSED AND WITHOUT MOUNTING UNITS.
- METHODS OF SUPPORT FOR THE STEELWORK THAT WILL PERMIT ATTACHMENT TO STRUCTURAL STEEL OR THROUGH BOLTS IN CONCRETE CONSTRUCTION SHOULD BE FAVORED. DO NOT USE CONCRETE OR MASONRY ANCHORS IN DIRECT TENSION.
- ALL UNITS THAT ARE WALL MOUNTED OR WALL SUPPORTED ARE TO BE PROVIDED WITH SUPPORTS WHERE NECESSARY. WALL SUPPORTS ARE TO BE SUPPLIED AND INSTALLED BY THE CUSTOMER OR HIS CONTRACTORS. SEE PLAN AND DETAIL SHEETS FOR SUGGESTED LOCATIONS AND MOUNTING HOLE LOCATIONS.
- ALL CEILING MOUNTED FIXTURES, AIR VENTS, SPRINKLERS, ETC. TO BE FLUSH MOUNTED, OR SHALL NOT EXTEND MORE THAN 6,35mm (1/4") BELOW THE FINISHED CEILING.
- CONTROL WALLS WITH TUBE HANGER PASSAGE ABOVE SHALL BE CONSTRUCTED TO 2130mm (7'-0") HIGH.
- FLOOR SLABS ON WHICH EQUIPMENT IS TO BE INSTALLED MUST BE LEVEL TO 3,17mm (1/8") in 3050mm (10'-0")
- DIMENSIONS ARE TO FINISHED SURFACES OF ROOM.
- CUSTOMERS CONTRACTOR MUST PROVIDE ALL PENETRATIONS IN POST TENSION FLOORS.
- CUSTOMERS CONTRACTOR MUST PROVIDE AND INSTALL ANY NON-STANDARD ANCHORING. DOCUMENTS FOR STANDARD ANCHORING METHODS ARE INCLUDED WITH GE EQUIPMENT DRAWINGS FOR GEOGRAPHIC AREAS THAT REQUIRE SUCH DOCUMENTATION.
- CUSTOMERS CONTRACTOR MUST PROVIDE AND INSTALL HARDWARE FOR "THROUGH THE FLOOR" ANCHORING AND/OR ANY BRACING UNDER ACCESS FLOORS. THIS CONTRACTOR MUST ALSO PROVIDE FLOOR DRILLING THAT CANNOT BE COMPLETED BECAUSE OF AN OBSTRUCTION ENCOUNTERED WHILE DRILLING BY THE GE INSTALLER SUCH AS REBAR ETC.
- IT IS THE CUSTOMER'S RESPONSIBILITY TO PERFORM ANY FLOOR OR WALL PENETRATIONS THAT MAY BE REQUIRED. THE CUSTOMER IS ALSO RESPONSIBLE FOR ENSURING THAT NO SUBSURFACE UTILITIES (E.G., ELECTRICAL OR ANY OTHER FORM OF WIRING, CONDUITS, PIPING, DUCT WORK OR STRUCTURAL SUPPORTS (I.E. POST TENSION CABLES OR REBAR)) WILL INTERFERE OR COME IN CONTACT WITH SUBSURFACE PENETRATION OPERATIONS (E.G. DRILLING AND INSTALLATION OF ANCHORS/SCREWS) PERFORMED DURING THE INSTALLATION PROCESS. TO ENSURE WORKER SAFETY, GE INSTALLERS WILL PERFORM SURFACE PENETRATION OPERATIONS ONLY AFTER THE CUSTOMER'S VALIDATION AND COMPLETION OF THE "GE SURFACE PENETRATION PERMIT"

SHEET TITLE: STRUCTURAL LAYOUT

MODALITY TYPE: REVOLUTION XQi

THIS PLAN IS SUBMITTED TO SUGGEST LOCATION OF GE HEALTHCARE EQUIPMENT. GE HEALTHCARE MAKES NO WARRANTY, REPRESENTATION OR GUARANTEE IN PREPARING THIS PLAN. EVERY EFFORT HAS BEEN MADE TO CONFORM DETAILS TO ACTUAL EQUIPMENT EXPECTED TO BE INSTALLED. IT IS NOT TO BE USED FOR ANY CONSTRUCTION OR INSTALLATION PURPOSES. GE HEALTHCARE ACCEPTS NO RESPONSIBILITY FOR ANY DAMAGES RESULTING THEREFROM.

PROJECT TITLE:

VA BROOKLYN
MEDICAL CENTER
BROOKLYN, NEW YORK

PROJECT	REVISION
111639	00

DATE: 01.Jul.11
DRAWN BY: CPC
CHECKED BY: REK

REVISION HISTORY:


SHEET

S1

GE Healthcare

IS Services Design Center

Wisconsin

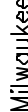

Milwaukee

This drawing is based on Sketch No.: 11mee047 VA Brooklyn Qxi revision P1M R3 RQ - 119540

THIS SHEET IS PART OF THE DOCUMENT SET LISTED ON SHEET C1 AND SHOULD NOT BE SEPARATED