


ELECTRICAL DETAIL  
CONDUITS THRU-FLOOR (TYPICAL)

ELEC-9

REV. DATE: 08/08/94

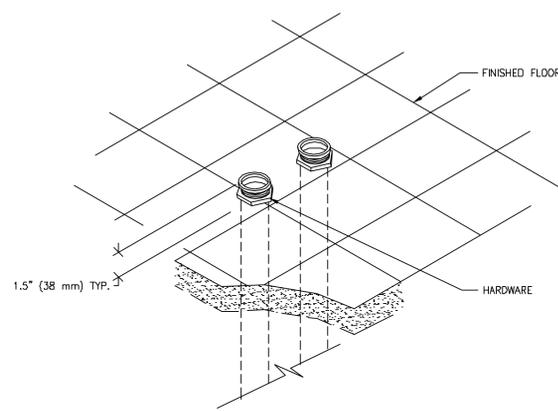


DETAIL NOT TO SCALE

ELECTRICAL DETAIL  
HORIZONTAL WALL DUCT (TYPICAL)

ELEC-5

REV. DATE: 03/19/04

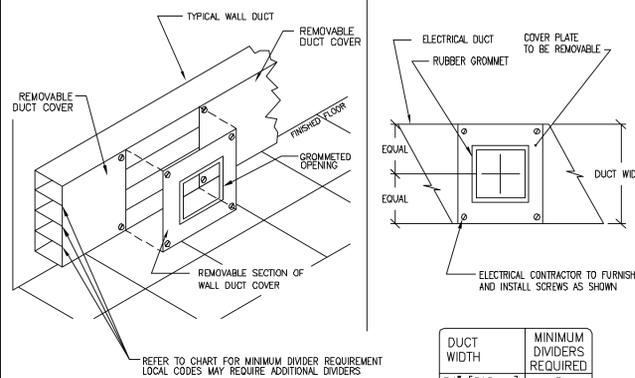


DETAIL NOT TO SCALE

DUCT WIDTH	MINIMUM DIVIDERS REQUIRED
24" [610mm]	2
18" [457mm]	2
10" [254mm]	2
6" [152mm]	1
4" [102mm]	1

ELECTRICAL DETAIL  
X-RAY MAIN DISCONNECT PANEL

ELEC-15

REV. DATE: 01/25/07

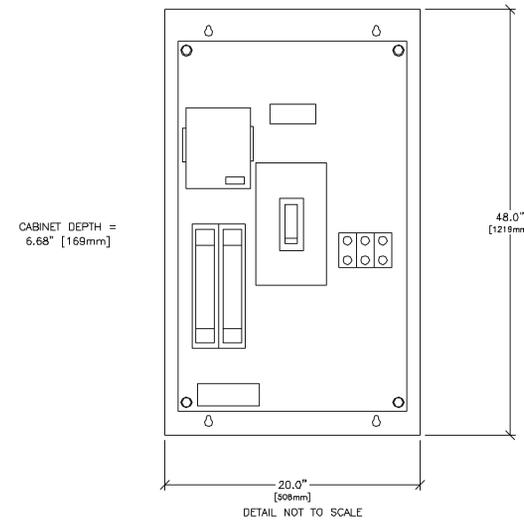


DETAIL NOT TO SCALE

ELECTRICAL DETAIL  
INSITE CONNECTION (TYPICAL)

ELEC-1

REV. DATE: 04/24/02

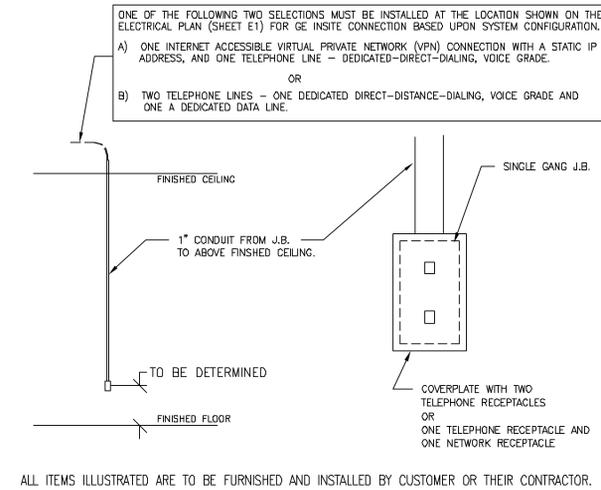


DETAIL NOT TO SCALE

ELECTRICAL DETAIL  
BOX WITH COVERPLATE AND NETWORK JACK

ELEC-83

REV. DATE: 10/06/98

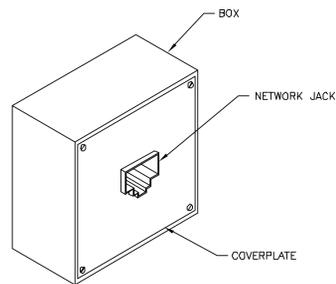


DETAIL NOT TO SCALE

ELECTRICAL DETAIL  
NETWORK CONNECTION (TYPICAL)

ELEC-84

REV. DATE: 03/06/04

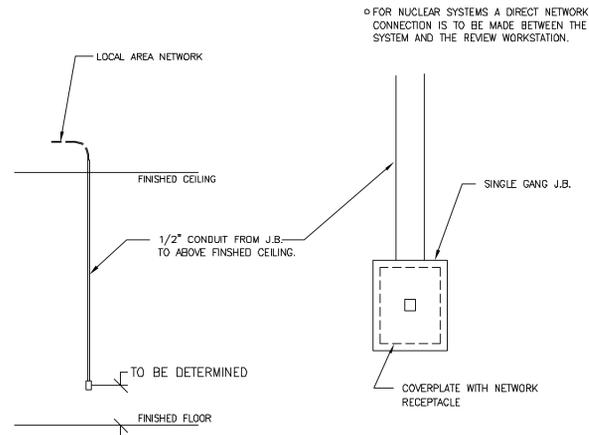


DETAIL NOT TO SCALE

ELECTRICAL DETAIL  
WARNING LIGHT DIAGRAM

ELEC-72

REV. DATE: 05/14/09

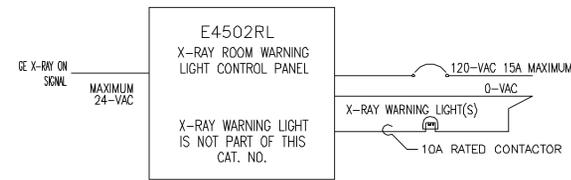


UNLESS SPECIFIED ON SHEET A1 AS BEING INCLUDED ON EQUIPMENT ORDER, ALL ITEMS ILLUSTRATED ARE TO BE FURNISHED AND INSTALLED BY CUSTOMER'S CONTRACTOR

DRAWING NOT TO SCALE

ELECTRICAL DETAIL  
EMERGENCY OFF BUTTON

ELEC-16

REV. DATE: 05/14/09

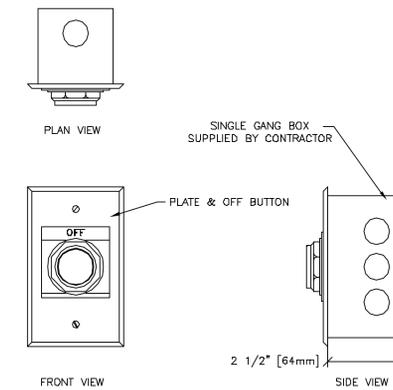

DETAIL NOT TO SCALE

**GE Healthcare**  
IS Services Design Center  
Milwaukee, Wisconsin

SHEET TITLE: ELECTRICAL DETAILS  
MODALITY TYPE: REVOLUTION XQI

THIS PLAN IS SUBMITTED TO SUGGEST LOCATION OF GE HEALTHCARE EQUIPMENT. IN PREPARING THIS PLAN, EVERY EFFORT HAS BEEN MADE TO CONFORM DETAILS TO ACTUAL EQUIPMENT EXPECTED TO BE INSTALLED. IT IS NOT TO BE USED FOR CONSTRUCTION OF EQUIPMENT UNLESS THE COMPANY CANNOT ACCEPT RESPONSIBILITY FOR ANY DAMAGES RESULTING THEREFROM.

PROJECT TITLE:  
**VA BROOKLYN  
MEDICAL CENTER**  
BROOKLYN, NEW YORK

PROJECT	REVISION
111639	00
DATE:	01.JUL.11
DRAWN BY:	CPC
CHECKED BY:	REK

REVISION HISTORY:


SHEET  
**E3**

RQ - 119540 PIM R3 This drawing is based on Sketch No.: 111mee047 VA Brooklyn QX1 revision