


EQUIPMENT ON ORDER FROM GE HEALTHCARE, INSTALLED BY GE HEALTHCARE, PER : NEITHER A QUOTE OR GON WAS ISSUED AT THE DATE OF THESE DRAWINGS				EQUIPMENT CROSS REFERENCE CHART			
NOTE: LOCAL CONDITIONS MAY DICTATE THAT ITEMS IDENTIFIED IN THIS CATEGORY BE INSTALLED BY OTHERS.				P = PREAPPROVAL C = CALCULATIONS/ S = SPECIFICATIONS ONLY			
				SEISMIC STATUS			
ITEM NO.	QUANTITY ORDERED	REFER TO SHEET "D"					
	ITEM DESCRIPTION (* = EXISTING/REINSTALL)	WEIGHT	HEAT OUTPUT (PER HOUR)	DETAIL NO.	STRC PLAN	ELEC PLAN	
①	* REVOLUTION XQ/1 TUBESTAND IN REGULAR CONFIGURATION	551 lbs	549 btu	B3525	B35 25A	TBS	C
②	* REVOLUTION XQ/1 DIGITAL CHEST READER	551 lbs	348 btu	B3525D	---	WLS	C
③	* OPERATORS CONSOLE	15 lbs	194 btu	A80066	---	SCC1	C
④	* ADS MONITOR AND KEYBOARD	52 lbs		B3525C	---		S
⑤	* ADS COMPUTER	136 lbs	1549 btu				
⑥	* REVOLUTION 65kw GENERATOR	848 lbs	1000 btu	A0720B	S71	SKL	C
⑦	* REVOLUTION XQ/1 DETECTOR SUPPORT ASSEMBLY	66 lbs	1706 btu	B0557D	---	DSA	-

--	--	--	--	--	--	--	--

EXISTING CEILING HEIGHT = 9'-2'

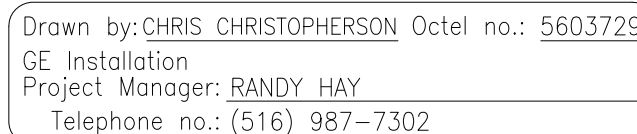

ITEM NO.	ITEM DESCRIPTION (* INDICATES EXISTING)
60	*MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 42 IN. W X 88 IN. H (1067mm X 2083mm), CONTINGENT ON A 96 IN. (2438mm) CORRIDOR WIDTH
61	**RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CATALOG: 800-200-9760, GE CAT. NO. WX1ABWW-DF-XIU
62	*DOOR LIMIT SWITCH (NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES)
63	COUNTER TOP FOR EQUIPMENT-MINIMUM DEPTH 30 IN. OR ADDITIONAL SHELVING MAY BE REQUIRED. PROVIDE OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RAYWAY BELOW COUNTER.
64	7'-0" HIGH CONTROL WALL WITH LEAD GLASS WINDOWS

90 \*X-RAY ROOM WARNING LIGHT CONTROL PANEL. REFERENCE JUNCTION POINT 'XRLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -E4502RL FOR WARNING LIGHT CONTROL ONLY

91 \*EMERGENCY OFF SWITCH (NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES)

92 \*MAIN DISCONNECT, REFERENCE JUNCTION POINT 'A' ON SHEET 'E1' FOR DETAILED DESCRIPTION. GE CAT. NO. E4502ST DR WITH AUTO DISCONNECT E4502RP.  
(20 W x 48 H x 6.69 D) (N.D.)

- THE REQUIRED CEILING HEIGHT INDICATED ON THESE PLANS IS TO ENSURE EQUIPMENT FUNCTION IS NOT INHIBITED. CONSULT WITH YOUR LOCAL GEHC IS SPECIALIST REGARDING ACCEPTABILITY OF OTHER CEILING HEIGHTS.
- CHECK ALL DOOR OPENINGS AND HALLWAYS FROM DELIVERY LOCATION TO WHERE EQUIPMENT IS TO BE INSTALLED TO ENSURE THE ROUTE PHYSICALLY AND STRUCTURALLY WILL ACCOMMODATE THE EQUIPMENT AS SHIPPED.
- RADIATION PROTECTION REQUIREMENTS ARE NOT INDICATED ON THIS PLAN. WHERE NEEDED PER NATIONAL OR LOCAL CODE THEY SHALL BE SPECIFIED BY A QUALIFIED RADIOLOGICAL PHYSICIST.
- THE DEVELOPMENT OF THE EQUIPMENT LAYOUT, ROOM DIMENSIONS, MECHANICAL AND ELECTRICAL SUGGESTIONS IS PRECIPITATED UPON THE BEST INFORMATION OBTAINABLE FROM THE SITE, COUPLED WITH THE CUSTOMER'S KNOWN DESIRES. ARCHITECTURAL OR ELECTRICAL CHANGES INCLUDING RELOCATION OF EQUIPMENT ILLUSTRATED ON THIS DRAWING IS ALLOWED ONLY WITH NOTIFICATION, IN WRITING, AND REVIEW BY GEHC. GEHC DEPARTMENT OF EQUIPMENT OPERATION, SERVICEABILITY, AND RESTRICTING CABLE LENGTHS, ETC., MAKE THIS ESSENTIAL FOR A PROPER IS. GEHC RESERVES THE RIGHT TO MAKE ON THE JOB CHANGES BECAUSE OF CUSTOMER REQUIREMENTS AND/OR OBSTACLES IN CONSTRUCTION, ETC..
- ALL WORK TO BE IN COMPLIANCE WITH NATIONAL AND LOCAL BUILDING SAFETY CODES.
- DIMENSIONS ARE TO FINISHED SURFACES OF ROOM

- AMBIENT OPERATING TEMPERATURE: 59 TO 75 DEGREES (F), MAXIMUM ALLOWABLE TEMPERATURE CHANGE OF 15 DEGREES (F)/HOUR.
- HUMIDITY: REFER TO PREINSTALLATION MANUAL FOR THE EQUIPMENT ILLUSTRATED ON THIS DRAWING.
- ALTITUDE: NOT TO EXCEED 8,000 FT. ABOVE SEA LEVEL.
- THE ENVIRONMENT FOR THE ELECTRONICS CABINET MUST BE CONTROLLED SO THE FOLLOWING RESTRICTIONS ARE NOT EXCEEDED:
- DO NOT RESTRICT THE AIR INTAKE AT THE LOWER FRONT OR AIR EXHAUST AT THE TOP OF THE ELECTRONICS CABINETS.

IMAGE INTENSIFIERS MUST BE LOCATED IN AMBIENT STATIC MAGNETIC FIELDS OF LESS THAN 1 GAUSS TO GUARANTEE SPECIFIED IMAGING PERFORMANCE.

X-RAY TUBES MUST BE LOCATED IN AMBIENT STATIC MAGNETIC FIELDS OF LESS THAN 10 GAUSS TO GUARANTEE SPECIFIED PERFORMANCE.

SYSTEM ELECTRONICS MUST BE LOCATED IN AMBIENT STATIC MAGNETIC FIELDS OF LESS THAN 10 GAUSS TO GUARANTEE DATA INTEGRITY.

OPERATORS CONSOLE EQUIPMENT MUST BE LOCATED IN AMBIENT STATIC MAGNETIC FIELDS OF LESS THAN 10 GAUSS TO OBTAIN SPECIFIED GEOMETRIC LINEARITY.

THIS PLAN IS SUBMITTED TO SUGGEST LOCATION OF GE HEALTHCARE EQUIPMENT AND ASSOCIATED APPARATUS, ELECTRICAL WIRING DETAILS AND ROOM ARRANGEMENTS. IN PREPARING THIS PLAN, EVERY EFFORT HAS BEEN MADE TO CONFORM DETAILS TO ACTUAL EQUIPMENT EXPECTED TO BE INSTALLED. IT IS NOT TO BE USED FOR ACTUAL CONSTRUCTION PURPOSES; HOWEVER, THE COMPANY CANNOT ACCEPT RESPONSIBILITY FOR ANY DAMAGES RESULTING THEREFROM.

VA BROOKLYN  
MEDICAL CENTER  
BROOKLYN, NEW YORK

### REVISION HISTORY

SHEET

A