

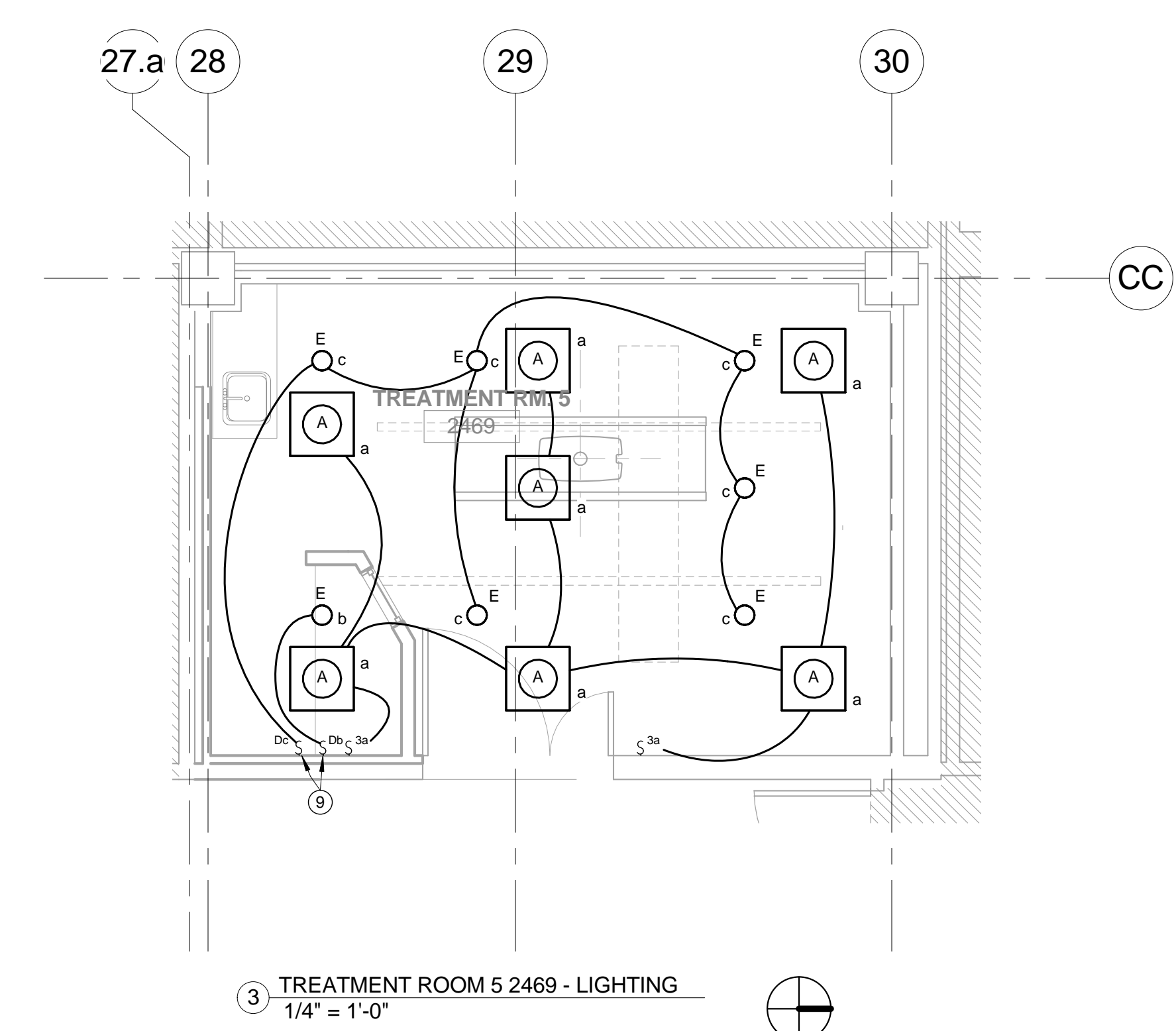
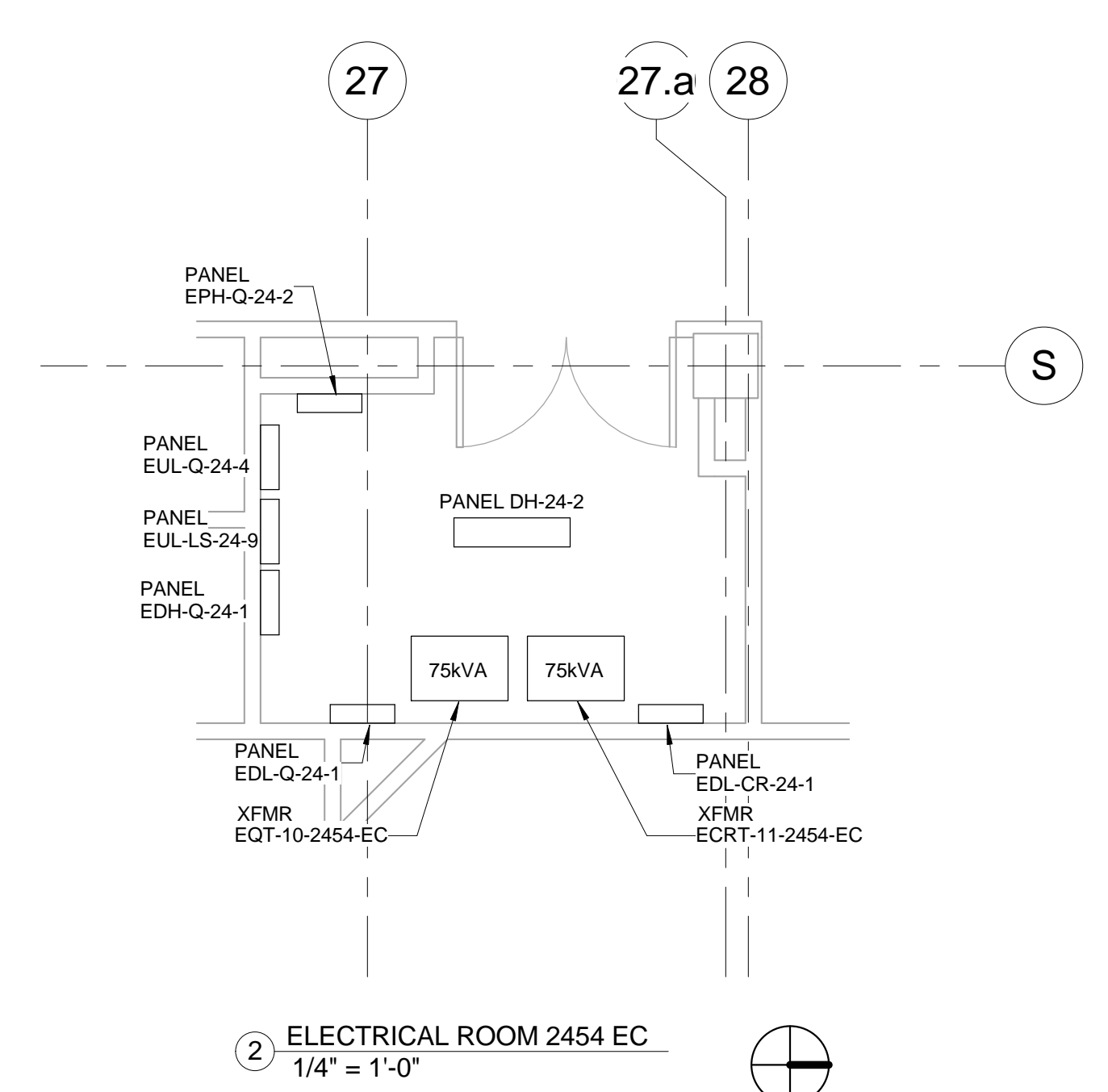


A
three inches = one foot
1
0
6"
B
one and one half inches = one foot
0
6"
C
one inch = one foot
0
6"
D
three quarters inch = one foot
0
6"
E
one half inch = one foot
0
6"
F
three eighths inch = one foot
0
6"
one quarter inch = one foot
0
6"
one eighth inch = one foot
0
6"

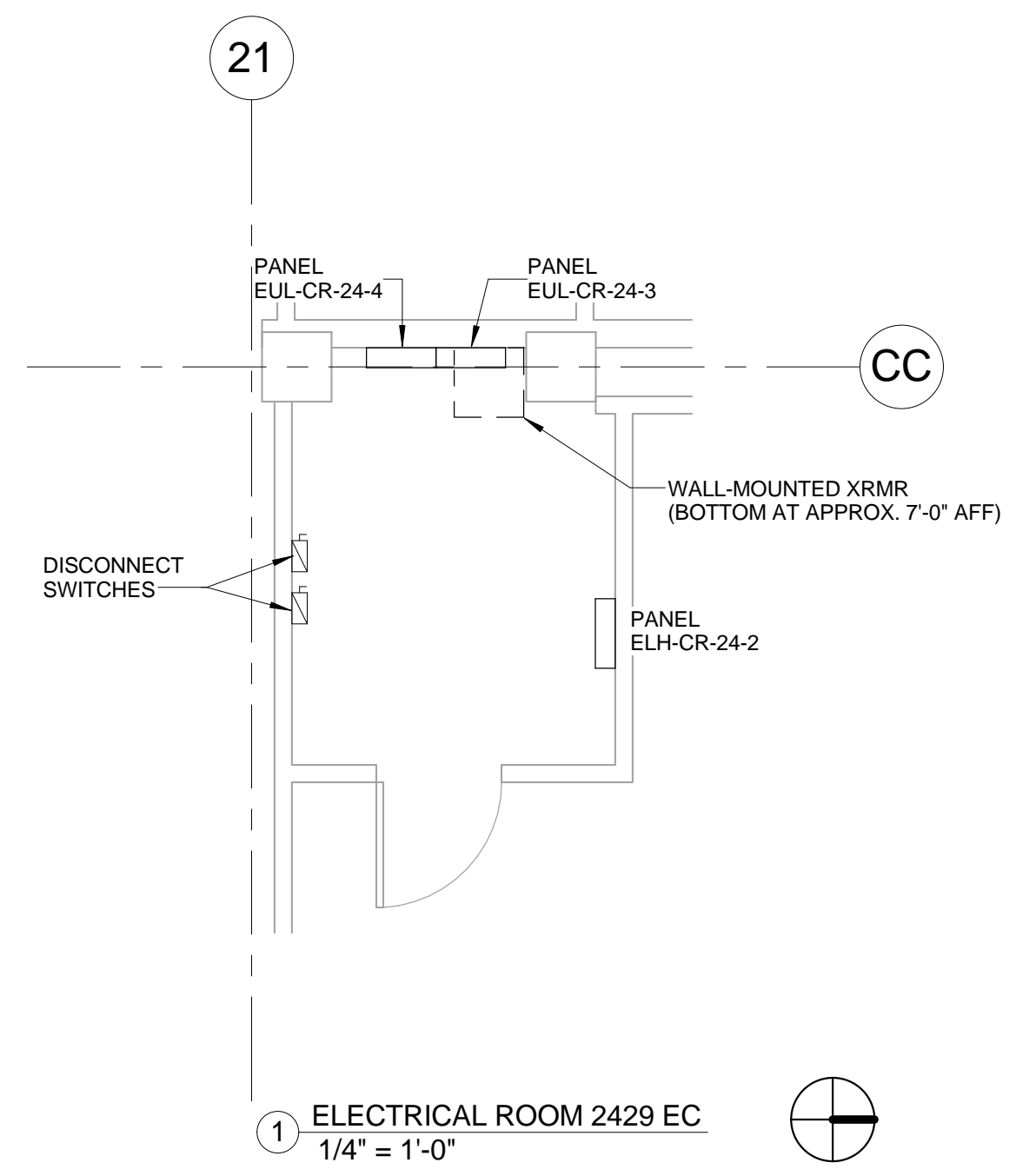

- ### GENERAL NOTES
- NOT ALL REQUIREMENTS INDICATED ON THESE DRAWINGS. REFER TO SIEMENS YS10 EQUIPMENT DRAWINGS FOR MORE REQUIREMENTS, INFORMATION, COORDINATION, AND DIMENSIONS. CONTRACTOR SHALL INCLUDE ALL SCOPE ITEMS FROM THESE DRAWINGS AND SIEMENS DRAWINGS IN BASE BID.
 - PROVIDE A CONDUIT MEASURING TAPE FISH LINE IN ALL CONDUITS IN TREATMENT ROOM 5 2469.

- ### KEYED NOTES
- FEEDER FOR NEW SIEMENS X-RAY. ROUTE TO INDICATED PANEL. REMOVE EXISTING SPARE BREAKER IN PANEL AND PROVIDE NEW 70A/3P BREAKER RATED 58kV. SEE DETAIL 2/ES-701 AND SHEET ES-301 FOR FEEDER ROUTING AND FOR LOCATION OF PANEL DH-24-2. SEE DETAIL 1/ES-601 FOR FEEDER SIZE.
 - APPROXIMATE LOCATION OF X-RAY 70A/3P ENCLOSED CIRCUIT BREAKER PROVIDED BY CONTRACTOR. PROVIDE SHUNT TRIP FOR EPO WIRING.
 - EPO PUSHBUTTON WITH CLEAR PLASTIC COVER. PROVIDE (2)#12, #12G, IN 3/4" C. EMT BETWEEN EPO BUTTONS AND ENCLOSED CIRCUIT BREAKER IN THIS ROOM.
 - JUNCTION BOX ABOVE CEILING FOR WARNING LIGHT RELAY. PROVIDE 24V RELAY AND #12 WIRING IN 3/4" C. PER 'AUXILIARY WIRING' DIAGRAM ON SIEMENS SHEET E-101.
 - PROVIDE 'X-RAY ON DO NOT ENTER' WARNING LIGHT ABOVE DOOR. WIRE WITH (2)#12, #12G, IN 3/4" C.
 - APPROXIMATE LOCATION OF VERTICAL WIRING DUCT ROUTED FROM ABOVE CEILING TO FLOOR. SEE SIEMENS DRAWINGS FOR MORE INFORMATION.
 - (3)#2, #2G, IN 1-1/4" C.
 - CIRCUIT TO EXISTING 120V CIRCUIT IN THIS AREA. CONTRACTOR SHALL ENSURE NOT MORE THAN THE EQUIVALENT OF (8) DUPLEX RECEPTACLES ARE ON A CIRCUIT. ADJUST CIRCUITING TO MULTIPLE LOCAL 120V CIRCUITS AS REQUIRED.
 - PROVIDE 0-10V DIMMER COMPATIBLE WITH FIXTURE. WIRE DIMMER TO FIXTURES PER MANUFACTURER'S RECOMMENDATIONS.

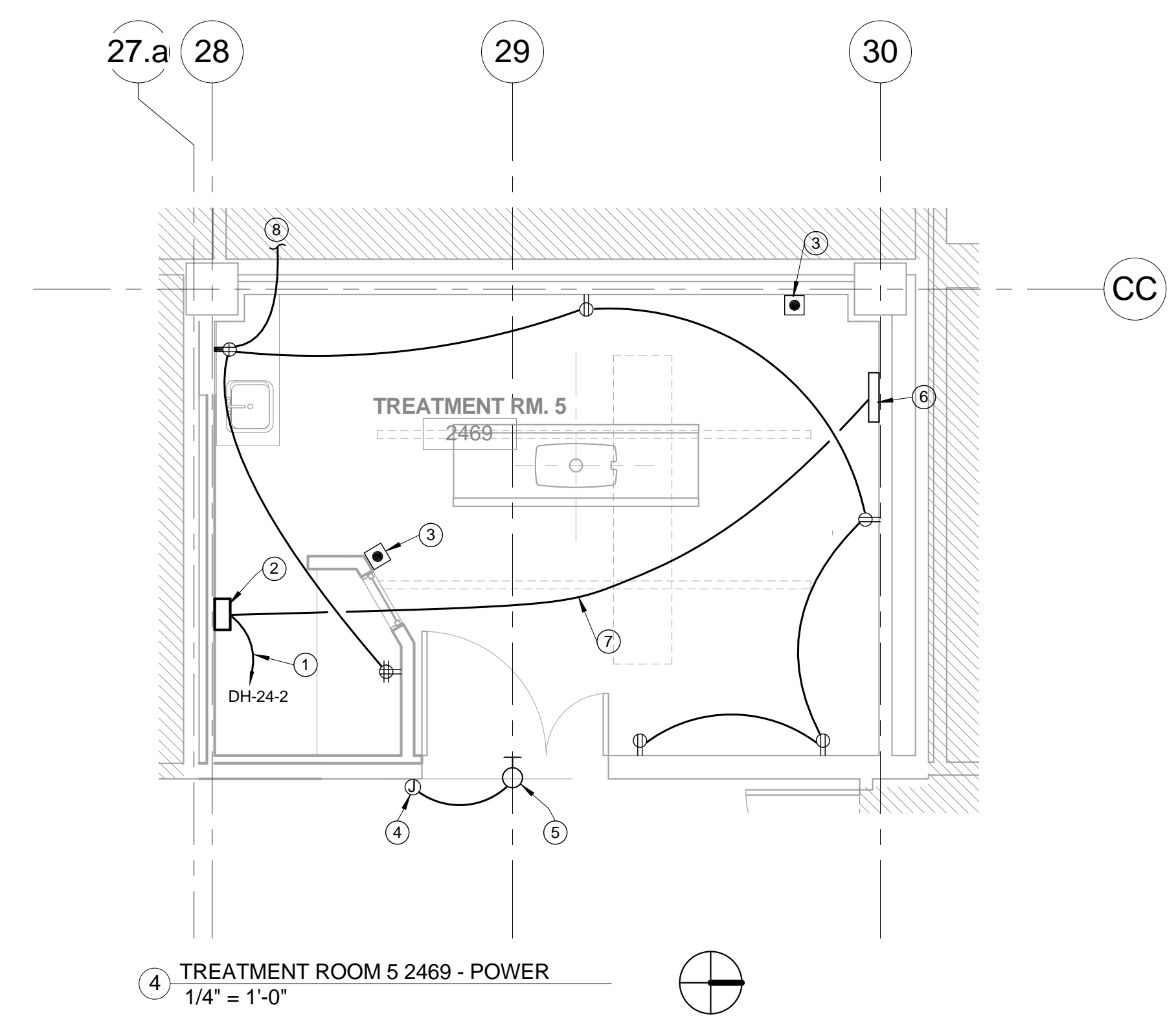

95% SUBMISSION - NOT FOR CONSTRUCTION

Revisions:	Date	Stamp / Seal	CONSULTANTS:	ARCHITECT/ENGINEERS:	Drawing Title	Project Name	Project Number	Office of Construction and Facilities Management	
			 9201 West Broadway Brooklyn Park, MN 55445 Tel: 763.315.7900 Fax: 763.315.7920	 Smith Seckman Reid, Inc. 11460 Tomahawk Creek Pkwy, Suite 320 Leawood, KS 66211 (913) 319-3560 FAX: (913) 319-3579 www.ssr-inc.com Project # 1254071	ELECTRICAL ENLARGED PLANS	111 RENOVATE X-RAY 5	695-12-195		
					Location	Building Number	Checked	Drawn	Date
					CLEMENT J. ZABLOCKI VA MEDICAL CENTER MILWAUKEE, WI	111	DGC	RDS	02.14.13
									Registration #
									15811
									ES-701
									Dwg. of

VA FORM 08-6231